

## PRODUCT FOCUS:

### ManDown Fall Sensor



TASK Community Care | Product Focus

## ManDown Fall Sensor



### Overview

The ManDown Sensor can be used as a safety device by a lone worker in a hazardous environment or as a fall detector for elderly or disabled people. It is also appropriate for individuals with motor control problems, such as those resulting from a stroke or Multiple sclerosis (MS).

The ManDown Sensor has a built-in tilt device, which will automatically send an alert to our 24/7 Monitoring Centre if the person wearing it is in a horizontal position, such as lying on the floor. Alternatively, pressing its central red button will activate a help call at any time.

### How it works

#### 1) Tilt Detection

When the sensor detects a tilt of more than 60 degrees, it will start its internal chronometer for approximately ten seconds. If the sensor comes back to a vertical position within these ten seconds no emergency alarm will be sent. If the sensor does not come back to a vertical position, a pre-alarm starts, accompanied by a beeping sound. This lasts approximately thirty seconds. The pre-alarm is cancelled if the sensor comes back to a vertical position or if the help call button is pressed.

#### 2) Help call button

Pressing the central red button of the ManDown Sensor will activate a help call at any time.

### Benefits

- ▶ Does not rely on strong impact to send an alert
- ▶ Particular effective for people with motor impairments who may have trouble pressing an emergency alert button
- ▶ Battery monitoring - optical indication by LED indicator

### Warranty

ManDown Fall Sensor – 1 Year Warranty

*Part of the TASK CareLink™ Falls Support Package, for people who have a substantial or critical need. Please contact us for further details on this or other TASK CareLink Call Packages*

**For more information on the ManDown Fall Sensor contact us at:**

TASK Community Care  
Unit 1, Block 3  
CityNorth Business Park  
Stamullen  
County Meath  
IRELAND

[info@taskltd.ie](mailto:info@taskltd.ie)  
Telephone: +353 (0)1 8435889  
[www.taskltd.ie](http://www.taskltd.ie)

TASK Community Care  
The Innovation Centre  
Northern Ireland Science Park  
Queen's Road  
BELFAST  
BT3 9DT

[info@taskltd.com](mailto:info@taskltd.com)  
Telephone: 0845 304 5535  
[www.taskltd.com](http://www.taskltd.com)